



Products for Clinical



Endo Clean by M Dent (17% Solution EDTA) 200 ml

Description Use to complete the removal of the smear layer and dentin debris

Indications to use Irrigation with 2 ml of 17% EDTA for one minute followed by a final rinse with NaOCl to remove the smear layer.



Non Latex Dental Dam 5"x 5" / 6"x 6"

Description A rubber dam is a rectangular sheet of latex used for root canal treatment. ** For isolate the tooth being treated from its environment, in particular from the bacteria in the oral cavity.

Indications to use The rubber dam is prepared by stretching the dam material over the frame and punching the appropriate number of holes in the dam material, as described above. The holes are stretched over the teeth so they poke through the rubber dam.



M Dent Blue

Description Detect fracture lines of cracked tooth. (Ready for Use)

Indications to use 1. Applied the Methylene Blue to the occlusal surface of the tooth with a paintbrush.
2. Ask the patient to bite on tooth sloth.
3. Diagnose the Crack.



Teeth Saver Set by M Dent

Description Used for soak tooth. To keep the tooth alive, it nourishes the cells until re-implantation

Indications to use 1. Add sterile purified water up to the marked line.
2. Shake bottle until media completely dissolved in the solution.
3. Put the tooth gently into the media, store in 4°C
Instruction for keeping the avulse tooth in the media
See dentist in nearby location immediately or within 1 hour, so PDL cells will be maintained as normal.




MU grafting tool

Description A tool used in grafting, especially Designed for periodontal regeneration based on cell biology



CATALOG

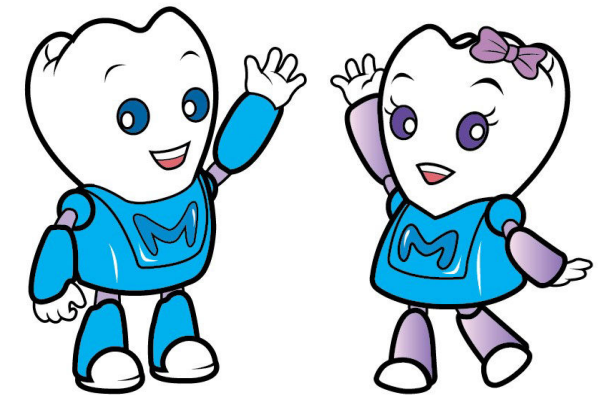

FACULTY OF DENTISTRY MAHIDOL UNIVERSITY

6 Yothi Rd., Payathai Rajthevi

Bangkok 10400


THAILAND

Phone: +66 2 2007631

Fax: +66 2 2007698

Website : www.dt.mahidol.ac.th/mdentshop

E-Mail : peerapong.tua@mahidol.ac.th

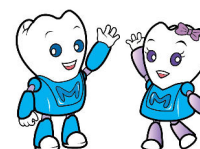

Products for Oral Health
Products for Clinical Treatment

Products for Oral



**Toothpaste 100 g
Soft Cool by M-Dent**



Toothbrush soft by M-Dent



**M-Dent package Toothbrush
and Toothpaste 40 g**



**Fluoride Mouthwash 0.05% 250/500 ml
(Mint)**

Products for Clinical

Erythrosine 6% 15 ml



Description Used to indicate areas of plaque build-up on teeth
Instructions 1. Dry the teeth with compressed air, retracting cheek and tongue. 2. Use small cotton pellet with cotton pliers to carry the solution to the teeth. 3. Apply solutions to the crowns of the teeth only. 4. Direct the patient to spread the agent over all surfaces of teeth with the tongue. 5. Examine the distribution of the agent and request the patient to rinse if indicated

Calcium Hydroxide 50 g



Description Used for eradication of microorganisms in root canal and for vital pulp therapy.
Instructions Mixed with water and put in the root canal.

Camphophenol 15 ml



Description Antibactericidal agent for root canal therapy.
Instructions Apply to the pulp or at the end of the root canal using a small cotton pallet or paper-point.

Cavity Varnish 5% 15 ml



Description Used to seal dentinal tubules to help prevent micro-leakage. Placed in a cavity preparation, which is to receive an amalgam, composite resin, or glass ionomer restoration, before a base is placed.
Instructions The cavity varnish is applied to the pulpal area, walls of the cavity preparation, and onto the edge of the margins of the preparation using a small cotton pallet. A second layer may be applied to fill voids. After liners and bases are placed into the cavity preparation, the tooth is permanently restored.

Clove Oil 15 ml



Description Used to Relieve tooth pain.
Instructions apply to the pulp or at the end of the root canal with a small cotton pallet or a paper-point.

Disclosing Paste 25 g



Description Used for diagnosing high-pressure areas with dentures at insertion and denture adjustment appointments.
Instructions 1. Paint a thin layer of paste over the entire tissue surface of both dentures using the brush provided 2. Place the dentures in the patient's mouth. Have the patient close on two cotton rolls, one on the right and one on the left for two minutes.

Root Canal Cement (RCT) 50 g



Description For obturation of root canal and cementation of gutta-percha during root canal therapy.
Instructions After successful removal of the pulp, the root canal is disinfected with the help of intracanal medicaments like Clove oil. obturation of root canal is finally done with root canal powder and gutta percha.
Keep out of reach of children

Talbot's Solution 15 ml



Description For application to the gums after scaling.
Instructions Wet a cotton roll with the solution. Apply to The gums after scaling.

Zinc Oxide 50 g



Description Used to Polish amalgam restoration.
Instructions Mix Zinc oxide with water to form a paste. polish the restoration with the help of a rubber cup.

Chlorhexidine 2% w/v 300 ml



Description Chlorhexidine gluconate is an effective broad-spectrum antimicrobial agent. Used as a soft tissue disinfectant pre-and post-surgery.

Xerostomia 300 ml



Description Used for relief of dryness of mouth and throat.
Instructions Rinse before meals 4 times a day (morning, noon, evening and before bed)

Chlorhexidine Mouthwash 0.12% 250/500 ml



Description Used for control of dental plaque and bacteria after periodontal surgery
Indications to use after meals 2 times a day (morning and evening) for 1 minute, 10-20 ml.

Special Mouthwash Conc. 30 ml



Description Used as an antibactericidal mouthwash.
Indications to use Drops into a glass of water. Mix properly and rinse with the solution.